
Handgelenkbandage
Wrist Bandage
Bandage de poignet
Polsiera
Bandáž na zápěstí
Elastyczna opaska
stawu nadgarstkowego
Csuklóvédő
El bileği bandajı
Bandáž na zápästie

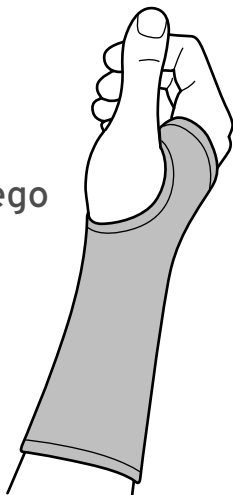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en Instructions for use

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sk Návod na použitie

de Liebe Kundin, lieber Kunde!

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en Dear Customer

This leaflet contains important information. Read it carefully to ensure that you can make the most of this product.

If you pass this article on to someone else, remember to give them these instructions for use as well.

Your Tchibo Team

Product numbers: **369 542 S**
 369 543 M
 369 544 L

The wrist bandage complies with directive
93/42/EEC for medical products.

Intended use

The wrist bandage is intended for use as an aid during physical activity and sport to provide support, gentle stabilisation and relief to the wrist and surrounding area.

Indications / contra-indications

The bandage can be used to provide support in the event of joint pains (e.g. joint degeneration), after wrist injuries (e.g. pulled muscles) and during physical exertion, including sports activities. The material is hypoallergenic.

The bandage must not be used, except on medical advice, for complex ligament instabilities, external injuries, circulatory disorders, water retention (oedema) or complex pain syndromes in the region of the hand, etc.

Danger to infants

Keep the bandage and packing material out of the reach of infants. There is a danger of suffocation!

For your safety, note the following:

Only use the bandage for its intended purpose. The bandage is not suitable for people who are allergic to any of the materials used or for people who are unable to make known that the bandage is uncomfortable.

For injuries, ask your doctor when and for how long you should wear the bandage.

If the bandage is too tight (for example, you experience a prickling sensation in your hand) or feels unpleasant, remove it immediately.

Only wear the bandage when awake. It might roll up as you sleep and restrict the blood supply.

Do not wear the bandage for longer than two hours continuously or for longer than six hours a day.

Do not apply any ointments or similar when wearing the bandage.

Care

Wash the bandage before using it for the first time and at least once every second to third wearing.

This bandage is machine washable. Follow the instructions on the sewn-in label.

Disposal

If you wish to dispose of the product, do so in accordance with current environmental regulations.

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Tchibo GmbH, Überseering 18,