Kühlpacks | Cool packs | Packs de froid | Chladivé obklady

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Only use the product as described in these instructions to avoid accidental injury or damage. Read these instructions carefully and keep them for future reference. If you give this product to another person, remember to also include these instructions.

Safety

Intended use

The cool packs are reusable and are designed to cool and soothe parts of the body. The cool packs are designed for private use and are not suitable for commercial, therapeutic or medical purposes.

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Do not use the cool packs on the body at temperatures below +5 °C. Do not use the cool packs on injured skin, in cases of Raynaud's syndrome or after taking pain-relieving medication. Not suitable for heat application. Do not heat the cool packs in the oven or microwave!

DANGER to children

• This product is not a toy. Suitable for children aged 3 and above, only under adult supervision. Store out of the reach of children.
• **Do not use as an aid to soothe teething infants! Make sure that children do not bite or chew on the cool packs.** The gel in the cool packs is not toxic. However, if your child develops any

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symptoms or experiences any discomfort, consult a doctor.
• Keep packaging materials out of the reach of children. They pose several risks, including the risk of suffocation!

WARNING - risk of hypothermia

• The product must not be used by people who are unable to feel heat or cold properly. Young children are more sensitive to heat and cold. Certain illnesses, such as diabetes, may also be accompanied by the inability to correctly perceive temperature. Frail or sick people or those with special needs or disabilities may not be able to call for help if the treated area feels too cold. Therefore, do not use this cooled product on children under 3 years of age, frail or sick people,

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or people with impaired sensitivity to cold. If you are in any doubt, consult your doctor before using this product.
• Cool the cool packs in the fridge and not in the freezer. Cool packs that have been cooled in the freezer could cause frostbite after just a few minutes of use!
• Never place the cool packs directly on your skin; always use a towel to protect your skin.
• For cooling treatments on the head, the product must not be too cold.
• Do not leave the cooled cool packs on for longer than approx. 10 minutes at a time. Risk of hypothermia!
• After using the cool packs, give your body sufficient time to recover before using them again.

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WARNING - damage to health

• The cool packs are intended for external use only. Their contents must not be ingested. If skin comes into contact with the contents - e.g. if the film becomes damaged - wash the parts of the skin that are affected under running water.
• Do not continue to use a damaged cool pack.

CAUTION - material damage

• Do not apply pressure to the cool packs.
• Keep sharp and pointed objects away from the cool packs.
• Overheating and extreme overcooling (storage in the freezer) can damage the cool packs.

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Use

1. Place the cool packs in the refrigerator for at least 1 hour. Massage the cool packs thoroughly, using only light pressure. They can then be used immediately.
2. Wrap the cool packs in a protective towel and place them on the area to be cooled.

Cleaning / storage

• Wipe the cool packs with a damp cloth as required.
• Do not clean the cool packs with solvents (e.g. disinfectants).
• Store the cool packs at room temperature and out of direct sunlight. The cool packs may decrease in volume over time. Replace them if necessary.

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Disposal

For the sake of the environment: Dispose of the product in accordance with applicable laws and regulations. Contact your local authorities for more information.

Film: PVC
Leaves in gel: PET
Gel: water, glycerin, carboxymethylcellulose, denatonium benzoate (bitting agent), sodium benzoate

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